

Software Kit

QS-901PK

General

The QS-901PK software kit is for upgrading the system software of the SVM-7500 series bedside monitor.

Safety Information

- ⚠ DANGER:** A danger alerts the user to a hazardous situation which causes death or serious injury.
- ⚠ WARNING:** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
- ⚠ CAUTION:** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

Do not leave the software USB memory near the patient or in reach of children. Otherwise it may be swallowed and cause injury.

⚠ CAUTION

- When inserting or removing the USB memory, turn the power off.
- Do not get the USB memory wet.
- Do not bend or give a strong impact to the USB memory.
- Avoid exposing the USB memory to high temperature or high humidity.

⚠ CAUTION

The USB memory may be damaged by static electricity. Before handling the USB memory, discharge static electricity from yourself by touching the equipotential grounding terminal on the bedside monitor.

USB Memory Handling Precautions

- Never disassemble or modify the USB memory.
- Do not give impact to the USB memory by dropping or bending.
- Do not handle the USB memory while eating or drinking.
- Do not get the USB memory wet.
- Do not peel off the label of the USB memory or attach another label.
- Never use the USB memory in unspecified instruments.
- Keep the USB memory slot clean. If dust gets into the slot, the USB memory does not function.
- Be careful not to lose the USB memory.
- Do not expose the USB memory to direct sunlight or leave it in a high temperature place.
- Do not store the USB memory where corrosive gas is generated.
- Never modify the data in the USB memory.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
 SHANGHAI KOHDEN MEDICAL
 ELECTRONIC INSTRUMENT CORP.
 No.567 Huancheng Bei Road, Shanghai Comprehensive
 Industrial Development Zone, Fengxian District,
 Shanghai 201401, China
 Phone +86 21-5743-6998 Fax +86 21-5743-6939
<https://www.nihonkohden.com/>

 European Representative
 NIHON KOHDEN EUROPE GmbH
 Raiffeisenstrasse 10, D-61191 Rosbach, Germany
 Phone +49 6003-827-0 Fax +49 6003-827-599