

Disk Electrode

BE-910P

General

The BE-910P disk electrode is used for monitoring EEG by a bedside monitor. With the electrode attached on the patient, patient EEG waveform is derived. Please read this manual thoroughly prior to operation to ensure optimum performance.

Safety Information

⚠ WARNING A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

- Before performing defibrillation, check that the disk electrodes are properly attached to the patient and connected to the bedside monitor. Touching the metal parts of disconnected disk electrodes may cause serious electrical shock or injury by discharged energy.
- Do not use this electrode during MRI examination because it may cause skin burn on the electrode placement area.

⚠ CAUTION

- Do not attach the electrode on a wound or inflamed skin.
- Do not use this electrode for electric stimulation.
- Do not store this electrode in an atmosphere with sulfur. It will deteriorate the electrode.

⚠ CAUTION

- Do not clean the electrode with hot water or chloric bleaching agent.
- Do not sterilize.
- Do not immerse the connector in liquid.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
1-31-4 Nishiochiai, Shinjuku-ku,
Tokyo 161-8560, Japan
Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, 61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 22 Apr 2019
2nd Edition: 16 Feb 2023

